# **Delta Basket**

## **Instructions for Use**

**EN**

**Delta Basket**

Instructions for use

**ES**

**Cesta Delta**

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**IT**

**Cesto Delta**

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# **Delta Basket**

## **DEVICE DESCRIPTION**

The Delta basket is a stone extracting device with a rigid shaft and an expandable basket of wire made of Nitinol and shaft made of stainless steel. It is designed to extract urinary stones from the calices of kidney in PCNL procedure. It is introduced through the working channel of the rigid nephroscope.

## **INTENDED PURPOSE & DURATION**

Delta Basket is intended to be used with rigid nephroscope for extraction of stones from calices for a period of less than 60 minutes.

## **CONTRAINDICATIONS**

None known

## **WARNINGS AND PRECAUTIONS**

- Manipulation of the device needs ultrasound or fluoroscopic guidance.
- If stone size is too large, break the stone into fragments and then retrieve, otherwise it may cause damage to the basket.
- The device is conductive. Avoid contact with electrified instrument.
- Do not use if product is damaged or expired and packaging integrity is compromised
- Do not reuse the device. Re use/ Resterlization of the device can compromise structural integrity of the device thereby causing serious harm to the patient. Reuse can contaminate the device causing serious harm to the patient by way of infection.
- Device use is intended to be carried out by trained physicians and technicians who were trained in endoscopic urological procedures. Standard endoscopic procedures should be employed.
- Do not use excessive force to manipulate the device, it may cause damage to the device.
- Not intended for permanent use. Only intended for transient use.

# **Delta Basket**

## **INSTRUCTIONS FOR USE**

- During PCNL procedures, after access, dilatation and lithotripsy baskets will be used for retrieval of stones from the system.
- The Basket should be passed through the working channel of scope.
- By pressing the U shape handle the basket will open and catch the object on releasing the handle.
- Carefully retrieve the object through the scope or along with the scope.

## **SUPPLY AND STORAGE**

The device is packed with dispenser and then packed in peel open Tyvek pouch made of polyethylene and Tyvek. Supplied sterilized by Ethylene Oxide gas in peel open package.

Device shall be stored at temperature range from 20<sup>0</sup>C to 40<sup>0</sup>C and away from direct sunlight and other adverse environmental conditions.

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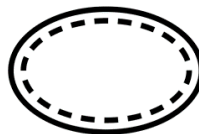
**Non-pyrogenic**

Apirógeno  
Apirogeno

**Single sterile barrier system with protective package inside**

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Blueneem, the Blueneem Logo are Registered and Trademarks of Blue Neem Medical Devices Private Limited Company.

© 2024 All rights reserved

## **BLUE NEEM MEDICAL DEVICES PRIVATE LIMITED**

Plot Nos 270 & 271, Road No 5,  
Harohalli Industrial Area, II Phase,  
Kanakapura Taluk, Ramanagara -  
562112, India.

Phone: +91 9739972855 / 54

E-Mail: [contact@blueneem.com](mailto:contact@blueneem.com)

Website: [www.blueneem.com](http://www.blueneem.com)

---

**Authorized Representative in the  
European Community:**

**Arazy Group (Ireland) Ltd**

19 Baggot Street Lower,  
Dublin 2, D02 X658, Ireland

E-mail: [Ireland@arazygroup.com](mailto:Ireland@arazygroup.com)