N Quadra[®] – Nitinol Stone Baskets

Instructions for Use

EN **N Quadra[®] – Nitinol Stone Baskets**
Instructions for use

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N Quadra® – Nitinol Stone Baskets

DEVICE DESCRIPTION

N Quadra®, N Quadra® Z, N Quadra® Helical, N Quadra® Helical Z, N Quadra® Flexible are four wire nitinol stone baskets available with adjustable handle which allows controlled assured basket expansion and contraction in a range of French sizes, lengths and basket diameters.

Blueneem Baskets are available in flexible and semi rigid working length configurations with variable basket diameters . Please refer to the product label for product specifications.

INTENDED PURPOSE & DURATION

Stone Basket is intended to be used in an endoscopic procedure for removal of urinary stones or foreign objects from the Ureter or the Kidney for a period of less than 60 minutes.

INDICATIONS FOR USE & CONTRAINDICATIONS

Indicated to be used to retrieve urinary stones or foreign objects form ureter or kidney. There are no known absolute contraindications but following is listed but not limited.

- Do not use the basket to grip the stone when its size is too large to remove endoscopically.
- To avoid stone impaction fluoroscopy or x-ray should be used to determine the size of the stone.

POTENTIAL COMPLICATIONS

Complications may occur at any time during or following the procedure. Possible complications include: Oedema, Urosepsis, Perforation of the ureter & Iatrogenic related infections.

WARNINGS

- Manipulation of the device needs ultrasound or fluoroscopic guidance.
- If stone size is too large, break the stone into fragments and then retrieve, otherwise it may cause damage to the basket.

N Quadra® – Nitinol Stone Baskets

- To avoid stone impaction, fluoroscopy or X-Ray should be used to assess the size of the stone.
- The device is conductive. Avoid contact with electrified instrument.
- Do not use if product is damaged or expired.
- Do not use if package integrity is compromised.
- Do not reuse the device. Re use/ Resterlization of the device can compromise structural integrity of the device thereby causing serious harm to the patient. Reuse can contaminate the device causing serious harm to the patient by way of infection.
- Do not open the basket or operate the device handle when the tip of the opening side of the basket is still inside the endoscope. It may lead to failure of the device.
- For semi rigid baskets do not bend the working length of the basket. As it may lead to kink or break the device.

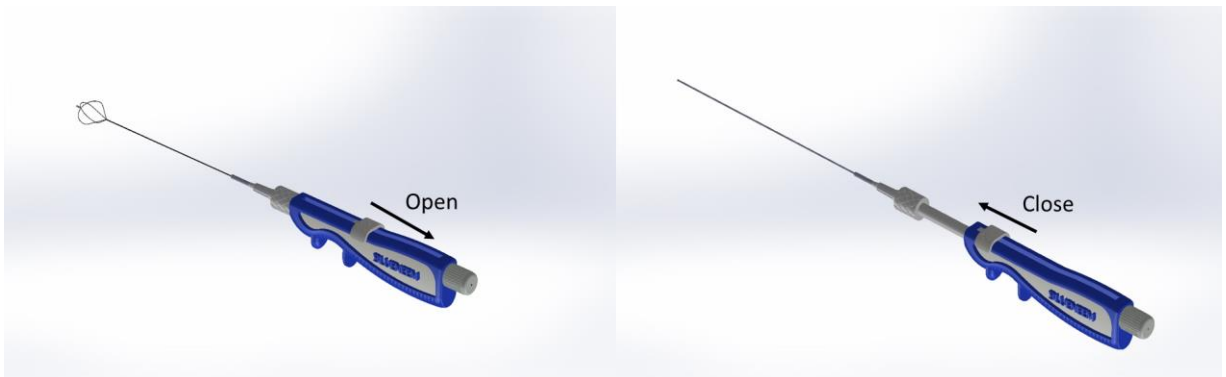
PRECAUTIONS

- Device use is intended to be carried out by trained physicians and technicians who were trained in endoscopic urological procedures. Standard endoscopic procedures should be employed.
- Do not use excessive force to manipulate the device, it may cause damage to the device.
- Not intended for permanent use. Only intended for transient use.

INSTRUCTIONS FOR USE

- Remove the stone basket from outer package and move the handle position to “Close” position to close the basket as shown in below figures.
- Insert the device into the scope and under direct visualization, locate the stone and advance the basket to the location just proximal to the stone.
- Slide the handle lever to open position as shown in below figures for different types of handles to deploy the basket near the stone.
- Maneuver the device until the stone is within the basket
- Once correctly positioned, draw the basket down around the stone and operate the handle as shown in below figures.
- Maintaining position of the stone within the basket, retract the scope and the basket.

N Quadra® – Nitinol Stone Baskets

SUPPLY AND STORAGE

The device is packed with dispenser and then packed in peel open Tyvek pouch made of polyethylene and Tyvek. Supplied sterilized by Ethylene Oxide gas in peel open package.

Device shall be stored at temperature range from 20⁰C to 40⁰C and away from direct sunlight and other adverse environmental conditions.

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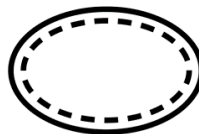
Non-pyrogenic

Apirógeno
Apirogeno

Single sterile barrier system with protective package inside

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Blueneem, the Blueneem Logo are Registered and Trademarks of Blue Neem Medical Devices Private Limited Company.

© 2024 All rights reserved

BLUE NEEM MEDICAL DEVICES PRIVATE LIMITED

Plot Nos 270 & 271, Road No 5,
Harohalli Industrial Area, II Phase,
Kanakapura Taluk, Ramanagara -
562112, India.

Phone: +91 9739972855 / 54

E-Mail: contact@blueneem.com

Website: www.blueneem.com

**Authorized Representative in the
European Community:**

Arazy Group (Ireland) Ltd

19 Baggot Street Lower,
Dublin 2, D02 X658, Ireland

E-mail: Ireland@arazygroup.com