Nephrostomy Balloon Dilator

Instructions for Use

EN

Nephrostomy Balloon Dilator

Instructions for use

ES

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Nephrostomy Balloon Dilator

DEVICE DESCRIPTION

Device package contains (contents may vary)

- Balloon dilator with syringe
- Balloon dilator with inflation device
- Balloon dilator with syringe and Amplatz sheath
- Balloon dilator with inflation device and Amplatz sheath

INTENDED PURPOSE & DURATION

Intended for fascial dilation, intended for stretching or enlarging the fascial tissue percutaneously prior to performing nephrostomy procedure up to a period of 60 minutes.

INDICATIONS FOR USE & CONTRAINDICATIONS

None known.

POTENTIAL COMPLICATIONS

UTI, False route & Bleeding.

WARNINGS AND PRECAUTIONS

- Always inflate the balloon with sterile liquid. Do not inflate the balloon with any kind of air or carbon dioxide. DO not over inflate, as excessive pressure may cause the balloon to rupture.
- If loss of pressure within the balloon occurs during inflation or if balloon ruptures during dilatation, immediately discontinue the procedure. Deflate the balloon carefully. Do not reinflate. Carefully withdraw the device if failure found during procedure.
- Do not exceed the maximum rated burst pressure (listed on label) for this balloon device. Refer label for device specifications.
- Use only compatible wire guide.
- Do not use if product is damaged or expired, reuse and resterilization of this device leads to loss in structural integrity and performance.
- This device is intended for use by trained physicians and this device should not be used without comprehensive knowledge of the indications, techniques and risks of the procedure.
- On completion of the procedure, dispose off the device as per local regulatory guidelines for bio-hazardous medical waste disposal.

Nephrostomy Balloon Dilator

PROCEDURE

Note: Using an aseptic technique, remove the device from its outer package and place it in sterile field.

DILATOR WITHOUT AMPLATZ SHEATH:

1. Perform initial access with puncture needle for the purpose of sequential dilation using standard procedure.
2. Remove the sleeve from the device and discard.
3. Lock the syringe on to the side arm stop cock of the balloon catheter and aspirate until all air is removed and close the stop cock.
4. Using imaging guidance, pass a guidewire of 0.035Inch to the desired distance of the tract to made.
5. Pass the balloon dilator over the guidewire and position the balloon with the help of radiopaque markers under imaging guidance, Please Ensure proper placement of the balloon at the site of dilation.
6. Fill the syringe or use the inflation device if provided, with contrast medium and eliminate air from the syringe or inflation device.
7. Open the stop cock on the dilator, connect the syringe or inflation device and inflate the balloon with the contrast medium. **CAUTION:** Inflate the balloon only with sterile medium. Do not inflate with air or any gas. To ensure proper regulation of balloon pressure, use of balloon inflation device with pressure gage is recommended. Do not exceed the recommended max pressure mentioned on the product label.
8. On reaching desired balloon pressure, close the stop cock and leave the inflated balloon at place for at least 30 seconds to allow adequate dilation.
9. Once desired dilation is achieved, open the stop cock and aspirate the liquid from the balloon using the syringe or inflation device.
10. Gently remove the dilator from the nephrostomy tract.

DILATOR WITH AMPLATZ SHEATH:

1. Remove the sleeve from the device and discard.
2. Lock the syringe on to the side arm stop cock of the balloon catheter and aspirate until all air is removed and close the stop cock.
3. Using fluoroscopic guidance, pass a guidewire of 0.035Inch to the desired distance of the tract to made.
4. Introduce the balloon dilator through the lumen of the amplatz sheath.
5. Pass the balloon dilator over the guidewire and position the balloon with the help of radiopaque markers under fluoroscopic guidance, Please Ensure proper placement of the balloon at the site of dilation.
6. Fill the syringe or use the inflation device if provided, with contrast medium and eliminate air from the syringe or inflation device.

Nephrostomy Balloon Dilator

7. Open the stop cock on the dilator, connect the syringe or inflation device and inflate the balloon with the contrast medium. CAUTION: Inflate the balloon only with sterile medium. Do not inflate with air or any gas. To ensure proper regulation of balloon pressure, use of balloon inflation device with pressure gage is recommended. Do not exceed the recommended max pressure mentioned on the product label.
8. On reaching desired balloon pressure, close the stop cock and leave the inflated balloon at place for at least 30 seconds to allow adequate dilation.
9. Once desired dilation is achieved, advance the previously loaded amplatz sheath onto the balloon to create the nephrostomy tract, after that open the stop cock and aspirate the liquid from the balloon using the syringe or inflation device.
10. Gently remove the dilator from the nephrostomy tract.

SUPPLY AND STORAGE

The device is packed with primary pack made of polyethylene and the secondary package is made of polyethylene and Tyvek pouch. Supplied sterilized by Ethylene Oxide gas in peel open package.

Device shall be stored at temperature range from 20⁰C to 40⁰C and away from direct sunlight and other adverse environmental conditions.

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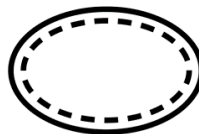
Non-pyrogenic

Apirógeno
Apirogeno

Single sterile barrier system with protective package inside

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Blueneem, the Blueneem Logo are Registered and Trademarks of Blue Neem Medical Devices Private Limited Company.

© 2024 All rights reserved

BLUE NEEM MEDICAL DEVICES PRIVATE LIMITED

Plot Nos 270 & 271, Road No 5,
Harohalli Industrial Area, II Phase,
Kanakapura Taluk, Ramanagara -
562112, India.

Phone: +91 9739972855 / 54

E-Mail: contact@blueneem.com

Website: www.blueneem.com

**Authorized Representative in the
European Community:**

Arazy Group (Ireland) Ltd

19 Baggot Street Lower,
Dublin 2, D02 X658, Ireland

E-mail: Ireland@arazygroup.com