Nephrostomy Drainage Set - Direct Puncture

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Nephrostomy Drainage Set – Direct Puncture

DEVICE DESCRIPTION

Nephrostomy Drainage Set – Direct Puncture is a kit comprising of flexible tube made of polyurethane, intended to be inserted into but not limited into the renal pelvis percutaneously and to gain access into the upper urinary tract by direct stick technique. The distal tip of the device has a pigtail shape for retention and metal trocar tip. The retention means have eye type ports to drain the fluids. For product specifications refer to the label.

Nephrostomy Drainage Set – Direct Puncture package includes(set components may vary):

- Radiopaque pigtail catheter (with or without Hydrophilic coating) (with or without locking mechanism)
- Metal cannula and metal trocar
- Plastic split sheath
- Urine bag connector
- Scalpel blade
- Catheter fixing disc

INTENDED PURPOSE & DURATION

The Nephrostomy Drainage Set Direct Puncture is intended to insert a PCN catheter percutaneously into the renal pelvis, and to gain access to the upper urinary tract to perform external drainage of urine from the renal pelvis by direct stick technique for a short-term period of up to 30 days.

INDICATIONS FOR USE & CONTRAINDICATIONS

Indicated when diagnosed to place drainage catheter percutaneously in kidney for renal drainage.

Device should not be used in the conditions of Bleeding Diathesis and uncontrolled hypertension, Anticoagulant use.

POTENTIAL COMPLICATIONS

- Bleeding (including chronic hematuria, post-procedural, intraprocedural, puncture site),
- Catheter dislocation/dislodgement,
- Catheter malposition/transcolonic catheter placement,
- Catheter occlusion/obstruction,
- Damage to adjacent organs,
- Urinary tract infection,
- Mechanical catheter damage and pain.

Nephrostomy Drainage Set – Direct Puncture

WARNINGS

- If a catheter has become malpositioned or if drainage ceases, the catheter should be promptly exchanged or removed.
- The Split-away Plastic Sheath, if retained, is not be used as introducer sheath for any purpose.
- Manipulation of the product needs ultrasound or fluoroscopy or other imaging guidance.

PRECAUTIONS

- Do not use if product is damaged or expired and do not reuse the device.
- Use only accessories supplied along with package.
- If the device is hydrophilic coated, activate the coat by using sterile water or saline or other isotonic solution to make the catheter insertion easier.
- Retain the plastic/metal cannulated straightener to straighten the catheter loop to remove the catheter.
- Dispose off the device and its components, if any, as per institutional and local regulatory guidelines for bio-hazardous medical waste disposal
- This device is intended for use by physicians trained and experienced in diagnostic and interventional techniques, standard techniques for placement of percutaneous drainage catheters should be employed.
- Periodic evaluation of the catheter is advised; catheter should not remain indwelled for more than 30 days after deployment.
- Do not forcefully advance any components during removal or placement of the catheter. Carefully remove the components if any resistance is occurred.
- Reuse and Re-sterilization of the device is strictly prohibited.
- Catheter shall be irrigated on routine basis to ensure function.
- Traction on the locking suture, If present, should be sufficient to ensure adequate retention of the tip, but should not be overly tight. Verify catheter tip configuration by fluoroscopy.
- While inserting a stiffening cannula into the catheter with retention suture, hold suture hub and manipulate it gently to avoid bunching or tangling of suture.

Nephrostomy Drainage Set – Direct Puncture

INSTRUCTIONS FOR USE

NOTE: Prior to use, If catheter is hydrophilic coated, immerse stent in sterile water or isotonic saline to allow the hydrophilic coat activated and become lubricious. This will allow the easy placement of stent under normal conditions.

STANDARD CATHETER-TYPE:

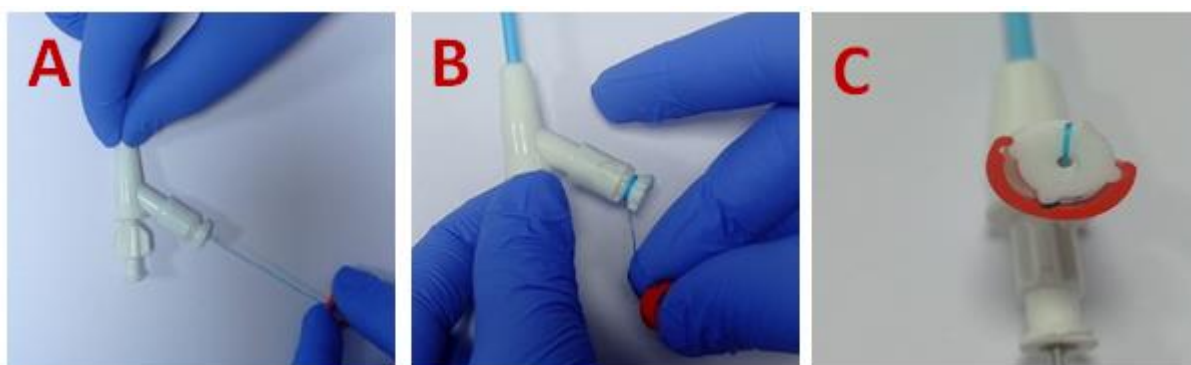
1. By retrograde pyelogram, ultrasound or CT scan, localize the renal collecting system to be drained.
2. After pyelogram, Identify and mark the skin site overlaying the collecting system. Anesthetize the marked skin site.
3. Use the peel away plastic sheath provided with the catheter to straight the pigtail of the catheter.
4. Then insert the metal straightener provided into the catheter and twist to lock with catheter hub, then insert the trocar needle through the metal straightener and twist to lock with metal straightener hub.
5. Remove the split away plastic Sheath on the catheter.
6. With the help of scalpel, make a small incision at the marked site.
7. Under fluoroscopic guidance, insert the catheter assembly through the incision. Advance the assembly to renal pelvis. Confirm placement via free flow of urine.
8. On confirmation, remove the straightener and trocar. The catheter will now acquire pigtail shape.
9. Secure the catheter to the catheter-fixing disc or to the skin.
10. Using the Urine bag connector, connect the catheter to the urine collection bag.

CATHETER WITH LOCKING MECHANISM-TYPE:

1. By retrograde pyelogram, ultrasound or CT scan, localize the renal collecting system to be drained.
2. After pyelogram, Identify and mark the skin site overlaying the collecting system. Anesthetize the marked skin site.
3. The product is supplied in ready to use state.
4. Use the split away plastic sheath provided with the catheter to straight the pigtail of the catheter.
5. Then insert the metal straightener provided into the catheter and twist to lock with catheter hub, then insert the trocar needle through the metal straightener and twist to lock with metal straightener hub.
6. Remove the Peel away plastic Sheath on the catheter.
7. With the help of scalpel, make a small incision at the marked site.
8. Under imaging guidance, insert the catheter assembly through the incision. Advance the assembly to renal pelvis. Confirm placement via free flow of urine.

Nephrostomy Drainage Set – Direct Puncture

- On confirmation, remove the straightener. The catheter will now acquire pigtail shape.
- After that pull the locking string as shown in image A.
- After pulling the string, wind the string firmly around the locking hub as shown in image B.
- Then lock the red clip to the locking hub, therefore the pigtail locks firmly inside the body as shown in image C.
- Traction on the locking string, if present, should be sufficient to ensure adequate retention of the tip, but should not be overly tight.
- Secure the catheter to the catheter-fixing disc or to the skin.
- Using the Urine bag connector, connect the catheter to the urine collection bag.

PROCEDURE: CATHETER REMOVAL FOR STANDARD TYPE

1. Detach the catheter from catheter fixing disc or skin.
2. Using the metal cannulated straightener supplied along with the catheter make the pigtail of catheter straight and remove from body carefully.
3. If necessary, place a guidewire, into the collecting system to maintain access while removing the catheter.
4. Remove the catheter gently and carefully.

CATHETER REMOVAL FOR LOCKING MECHANISM TYPE

1. For removal of Catheter from the body unlock the red clip from the locking hub of the catheter.
2. Unwind the string in anti-clockwise, after that turn the locking hub in anti-clockwise direction to release the locking of the string.
3. Using the metal cannulated straightener supplied along with the catheter make the pigtail of catheter straight and remove from body carefully.
4. If necessary, place a guidewire, into the collecting system to maintain access while removing the catheter.

Nephrostomy Drainage Set – Direct Puncture

SUPPLY AND STORAGE

The device is packed with primary pack made of polyethylene and the secondary package is made of polyethylene and Tyvek pouch. Supplied sterilized by Ethylene Oxide gas in peel open package.

Device shall be stored at temperature range from 20⁰C to 40⁰C and away from direct sunlight and other adverse environmental conditions.

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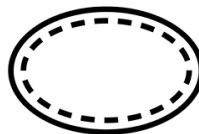
Non-pyrogenic

Apirógeno
Apirogeno

Single sterile barrier system with protective package inside

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Blueneem, the Blueneem Logo are Registered and Trademarks of Blue
Neem Medical Devices Private Limited Company.

© 2024 All rights reserved

BLUE NEEM MEDICAL DEVICES PRIVATE LIMITED

Plot Nos 270 & 271, Road No 5,
Harohalli Industrial Area, II Phase,
Kanakapura Taluk, Ramanagara -
562112, India.

Phone: +91 9739972855 / 54

E-Mail: contact@blueneem.com

Website: www.blueneem.com

**Authorized Representative in the
European Community:**

Arazy Group (Ireland) Ltd

19 Baggot Street Lower,
Dublin 2, D02 X658, Ireland

E-mail: Ireland@arazygroup.com