PCN Catheters

Instructions for Use

EN

PCN Catheters

Instructions for use

ES

Catéteres PCN

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IT

Cateteri PCN

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PCN Catheters

DEVICE DESCRIPTION

PCN Catheter is flexible tube made of polyurethane, intended to be inserted into but not limited into the renal pelvis percutaneously and to gain access into the upper urinary tract. The tip of the device has a pigtail shape for retention. The retention means have eye type ports to drain the fluids. Device packaging is provided with plastic/metal retention loop straightener, urine bag connector. For product specifications refer to the label.

PCN Catheters (Standard) package includes(set components may vary):

- Radiopaque pigtail catheter (with or without Hydrophilic coating)
- Plastic/metal cannulated loop straightener
- Plastic sheath
- Urine bag connector

PCN Catheters with locking mechanism package includes(set components may vary):

- Radiopaque pigtail catheter with locking mechanism (with or without Hydrophilic coating)
- Plastic/metal cannulated loop straightener
- Plastic sheath
- Urine bag connector

INTENDED PURPOSE & DURATION

PCN catheters are intended to be but not limited to inserted percutaneously into the renal pelvis and to gain access to the upper urinary tract to perform external drainage of urine from the renal pelvis for a short-term period of up to 30 days.

INDICATIONS FOR USE & CONTRAINDICATIONS

Indicated when diagnosed to place drainage catheter percutaneously in kidney for renal drainage.

No known contraindications.

Device should not be used in the conditions of Bleeding Diathesis and uncontrolled hypertension, Anticoagulant use.

POTENTIAL COMPLICATIONS

Potential complications include but not limited to Urinary tract infection, perforation of Kidney.

PCN Catheters

WARNINGS AND PRECAUTIONS

- If a catheter has become malpositioned or if drainage ceases, the catheter should be promptly exchanged or removed.
- The split away Plastic Sheath, if retained, is not be used as introducer sheath any purpose.
- Do not use if product is damaged or expired and do not reuse the device.
- Use only accessories supplied along with package.
- Manipulation of the product needs ultrasound or fluoroscopic guidance.
- If the device is hydrophilic coated, activate the coat by using sterile water or saline or other isotonic solution to make the catheter insertion easier.
- Retain the plastic/metal cannulated straightener to straighten the catheter loop to remove the catheter.
- Dispose off the device and its components, if any, as per institutional and local regulatory guidelines for bio-hazardous medical waste disposal
- This device is intended for use by physicians trained and experienced in diagnostic and interventional techniques, standard techniques for placement of percutaneous drainage catheters should be employed.
- Periodic evaluation of the catheter is advised; catheter should not remain indwelled for more than 30 days after deployment.
- Do not forcefully advance any components during removal or placement of the catheter. Carefully remove the components if any resistance is occurred.
- Reuse and Resterlization of the device is strictly prohibited.

PRODUCT RECOMMENDATIONS

For catheter loading Seldinger technique is employed. Recommended wire guide size is mentioned as follows:

Catheter size (in FR)	Accepted wire guide size (in Inch)
6, 7, 8 & 8.5	0.035
9, 10, 12, 14, 16 & 18	0.038

PCN Catheters

INSTRUCTIONS FOR USE

NOTE: Prior to use, If catheter is hydrophilic coated, immerse stent in sterile water or isotonic saline to allow the hydrophilic coat activated and become lubricious. This will allow the easy placement of stent under normal conditions.

STANDARD CATHETER-TYPE:

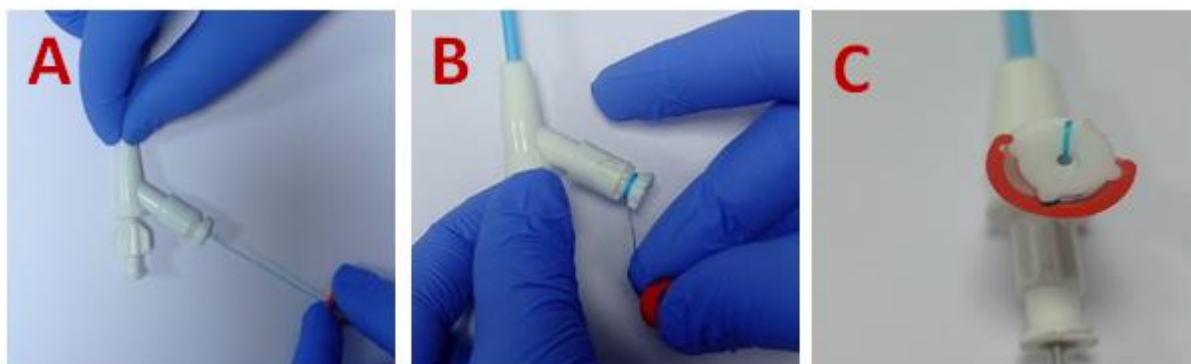
1. Perform procedure using standard procedures of puncture, compatible guidewire placement and dilatation.
2. Use the peel away plastic sheath provided on the catheter to straight the pigtail of the catheter and then insert the plastic or metal cannulated straightener provided and twist the hub to lock after that remove the peel away plastic sheath.
3. Then pass the pigtail end of the catheter over the external end of the guidewire.
4. Gradually advance the pigtail end well into the collecting system and remove the straightener.
5. Confirm the pigtail position using fluoroscopic guidance.
6. Holding the shaft of the catheter securely in position remove the guidewire carefully.
7. Secure the catheter to the catheter-fixing disc or to the skin.
8. Using the Urine bag connector, connect the catheter to the urine collection bag.

CATHETER WITH LOCKING MECHANISM-TYPE:

1. Perform procedure using standard procedures of puncture, compatible guidewire placement and dilatation.
2. The catheter is supplied in ready to use state.
3. Use the peel away plastic sheath provided with the catheter to straight the pigtail of the catheter.
4. Then insert the metal straightener provided into the catheter and twist to lock with catheter hub.
5. Remove the Peel away plastic Sheath on the catheter. When inserting the Straightener into the catheter with locking mechanism, hold the string during Straightener insertion to avoid bunching or tangling of suture. Luer lock should not be tighten along with Straightener, to do so may cause kinking of catheter.
6. Then pass the pigtail end of the catheter over the external end of the guidewire.
7. Gradually advance the pigtail end well into the collecting system and remove the Straightener.
8. Confirm the pigtail position using fluoroscopic guidance.
9. Holding the shaft of the catheter securely in position remove the guidewire carefully, Confirm placement via free flow of urine.

PCN Catheters

- On confirmation, remove the straightener. The catheter will now acquire pigtail shape.
- After that pull the locking string as shown in image A.
- After pulling the string, wind the string firmly around the locking hub as shown in image B.
- Then lock the red clip to the locking hub, therefore the pigtail locks firmly inside the body as shown in image C.
- Traction on the locking string, if present, should be sufficient to ensure adequate retention of the tip, but should not be overly tight.
- Secure the catheter to the catheter-fixing disc or to the skin.
- Using the Urine bag connector, connect the catheter to the urine collection bag.

PROCEDURE: CATHETER REMOVAL FOR STANDARD TYPE

1. Detach the catheter from catheter fixing disc or skin.
2. Using the plastic straightener supplied along with the catheter make the pigtail of catheter straight and remove from body carefully.
3. If necessary, place a guidewire, into the collecting system to maintain access while removing the catheter.
4. Remove the catheter gently and carefully.

CATHETER REMOVAL FOR LOCKING MECHANISM TYPE

1. For removal of Catheter from the body unlock the red clip from the locking hub of the catheter.
2. Unwind the string in anti-clockwise, after that turn the locking hub in anti-clockwise direction to release the locking of the string.
3. Using the plastic straightner supplied along with the catheter make the pigtail of catheter straight and remove from body carefully.
4. If necessary, place a guidewire, into the collecting system to maintain access while removing the catheter.

PCN Catheters

SUPPLY AND STORAGE

The device is packed with primary pack made of polyethylene and the secondary package is made of polyethylene and Tyvek pouch. Supplied sterilized by Ethylene Oxide gas in peel open package.

Device shall be stored at temperature range from 20⁰C to 40⁰C and away from direct sunlight and other adverse environmental conditions.

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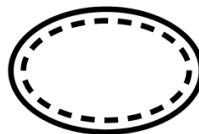
Non-pyrogenic

Apirógeno
Apirogeno

Single sterile barrier system with protective package inside

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Blueneem, the Blueneem Logo are Registered and Trademarks of Blue Neem Medical Devices Private Limited Company.

© 2024 All rights reserved

BLUE NEEM MEDICAL DEVICES PRIVATE LIMITED

Plot Nos 270 & 271, Road No 5,
Harohalli Industrial Area, II Phase,
Kanakapura Taluk, Ramanagara -
562112, India.

Phone: +91 9739972855 / 54

E-Mail: contact@blueneem.com

Website: www.blueneem.com

**Authorized Representative in the
European Community:**

Arazy Group (Ireland) Ltd

19 Baggot Street Lower,
Dublin 2, D02 X658, Ireland

E-mail: Ireland@arazygroup.com