Suprapubic Catheter

Instructions for Use

EN **Suprapubic Catheter**

Instructions for use

ES **Catéter suprapúbico**

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IT **Catetere sovrappubico**

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Suprapubic Catheter

DEVICE DESCRIPTION

Suprapubic catheter is a flexible drainage tube made of polyurethane with winged petals which enables retention. Device package includes :

- Radiopaque catheter retention winged petals,
- Metal cannula with metal trocar,
- Scalpel blade and
- Urine bag connector.

For product specifications refer to the label.

INTENDED PURPOSE & DURATION

Suprapubic catheter is intended to be inserted percutaneously into the urinary bladder and perform external drainage of urine from the urinary bladder. For a period of up to 30 days.

INDICATIONS FOR USE & CONTRAINDICATIONS

Indicated when diagnosed to place drainage catheter percutaneously in bladder for external drainage.

Device should not be used in the conditions where In patients with previous lower abdominal surgery or Patients with acute pelvic trauma and Absence of an easily palpable or ultrasonographically localized distended urinary bladder.

POTENTIAL COMPLICATIONS

Potential complications include but not limited to Urinary tract infection, Bladder perforation.

WARNINGS AND PRECAUTIONS

- Manipulation of device requires Ultrasound / Fluoroscopy / Other image guiding techniques.
- Do not use if product is damaged or expired.
- This device is intended for use by physicians trained and experienced in diagnostic and percutaneous techniques, standard techniques for placement of percutaneous drainage catheters should be employed. Use aseptic techniques for deployment.
- If drainage ceases, the catheter should be promptly exchanged or removed.
- Use only accessories supplied along with package and do not resterilize.
- On completion of the procedure, dispose off the device and its components, if any, as per institutional and local regulatory guidelines for bio-hazardous medical waste disposal.
- Periodic evaluation of the catheter is advised.
- Catheter should be irrigated on routine basis to ensure function.
- Do not reuse the device as it leads to potential complications.
- Not intended for permanent use.

Suprapubic Drainage Catheter Set

INSTRUCTIONS FOR USE

PROCEDURE: CATHETER PLACEMENT

1. Identify the puncture site appropriately and Anesthetize the area.
2. Advance the metal straightener into the catheter tip and secure with the lure lock to close the malecot wings, after that insert the Needle and secure to the hub.
3. If the bladder is not already distended by urine, fill the bladder with sterile saline.
4. Make a incision at the puncture site with the scalpel.
5. Insert the catheter into the bladder with the help of trocar needle. Observe for urine flow. Aspirate if necessary to confirm.
6. Make sure that the catheter is stabilized, remove the needle carefully and then release the luer lock and rotate the hub counter clockwise to open the malecot wings.
7. Secure the catheter to the catheter fixing disc or to the skin.
8. Attach the urine bag connector to the catheter and allow the bladder to empty.

PROCEDURE: CATHETER REMOVAL

1. Detach the catheter from catheter fixing disc or skin.
2. Gently pull the catheter carefully to remove.
3. If necessary, place a guidewire, into the bladder to maintain access.

SUPPLY AND STORAGE

The device is packed with primary pack made of polyethylene and the secondary package is made of polyethylene and Tyvek pouch. Supplied sterilized by Ethylene Oxide gas in peel open package.

Device shall be stored at temperature range from 20⁰C to 40⁰C and away from direct sunlight and other adverse environmental conditions.

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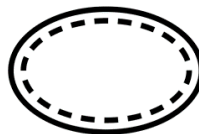
Non-pyrogenic

Apirógeno
Apirogeno

Single sterile barrier system with protective package inside

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Blueneem, the Blueneem Logo are Registered and Trademarks of Blue Neem Medical Devices Private Limited Company.

© 2024 All rights reserved

BLUE NEEM MEDICAL DEVICES PRIVATE LIMITED

Plot Nos 270 & 271, Road No 5,
Harohalli Industrial Area, II Phase,
Kanakapura Taluk, Ramanagara -
562112, India.

Phone: +91 9739972855 / 54

E-Mail: contact@blueneem.com

Website: www.blueneem.com

**Authorized Representative in the
European Community:**

Arazy Group (Ireland) Ltd

19 Baggot Street Lower,
Dublin 2, D02 X658, Ireland

E-mail: Ireland@arazygroup.com