Suprapubic Drainage Catheter Set

Instructions for Use

EN **Suprapubic Drainage Catheter Set**
Instructions for use

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Suprapubic Drainage Catheter Set

DEVICE DESCRIPTION

Suprapubic drainage catheterization set contains

Silicone foley catheter,

Plastic trocar with metallic needle tip,

Plastic peel away sheath and

Scalpel blade.

For product specifications refer to the label.

INTENDED PURPOSE & DURATION

Suprapubic Drainage Catheter Set is intended to create suprapubic access to the bladder and place a catheter for bladder drainage up to 07 days.

INDICATIONS FOR USE & CONTRAINDICATIONS

Indicated when diagnosed to place drainage catheter percutaneously in bladder for external drainage.

Device should not be used in the conditions where In patients with previous lower abdominal surgery or Patients with acute pelvic trauma and Absence of an easily palpable or ultrasonographically localized distended urinary bladder.

POTENTIAL COMPLICATIONS

Potential complications include but not limited to Urinary tract infection, Bladder perforation.

WARNINGS AND PRECAUTIONS

- Manipulation of device requires Ultrasound / Fluoroscopy / Other image guiding techniques.
- Do not use if product is damaged or expired.
- This device is intended for use by physicians trained and experienced in diagnostic and percutaneous techniques, standard techniques for placement of percutaneous drainage catheters should be employed. Use aseptic techniques for deployment.
- If drainage ceases, the catheter should be promptly exchanged or removed.
- Use only accessories supplied along with package and do not resterilize.
- On completion of the procedure, dispose off the device and its components, if any, as per institutional and local regulatory guidelines for bio-hazardous medical waste disposal.
- Periodic evaluation of the catheter is advised.
- Catheter should be irrigated on routine basis to ensure function.
- Do not reuse the device as it leads to potential complications.
- Not intended for permanent use.

Suprapubic Drainage Catheter Set

INSTRUCTIONS FOR USE

PROCEDURE: CATHETER PLACEMENT

1. Identify the puncture site appropriately and anesthetize the area.
2. If the bladder is not already distended by urine, fill the bladder with sterile saline.
3. Make a incision at the puncture site with the scalpel.
4. Insert the plastic trocar along with peel away sheath into the bladder and observe for urine flow to confirm the access.
5. Remove the plastic trocar leaving peel away sheath in place.
6. Through the peel away sheath, insert the drainage catheter into the bladder and inflate the balloon present at the catheter tip.
7. After inflation peel away the sheath using the holder available and remove leaving the catheter in the bladder.
8. Secure the catheter to the catheter-fixing disc or to the skin.
9. Attach the urine bag connector to the catheter and allow the bladder to empty.

PROCEDURE: CATHETER REMOVAL

1. Detach the catheter from catheter fixing disc or skin.
2. Deflate the balloon of the catheter.
3. Gently pull the catheter carefully to remove.

SUPPLY AND STORAGE

The device is packed with blister tray sealed with Tyvek pouch. Supplied sterilized by Ethylene Oxide gas in blister sealed tray.

Device shall be stored at temperature range from 20⁰C to 40⁰C and away from direct sunlight and other adverse environmental conditions.

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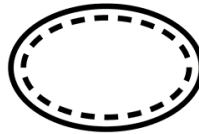
Non-pyrogenic

Apirógeno
Apirogeno

Single sterile barrier system with protective package inside

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Blueneem, the Blueneem Logo are Registered and Trademarks of Blue Neem Medical Devices Private Limited Company.

© 2024 All rights reserved

BLUE NEEM MEDICAL DEVICES PRIVATE LIMITED

Plot Nos 270 & 271, Road No 5,
Harohalli Industrial Area, II Phase,
Kanakapura Taluk, Ramanagara -
562112, India.

Phone: +91 9739972855 / 54

E-Mail: contact@blueneem.com

Website: www.blueneem.com

**Authorized Representative in the
European Community:**

Arazy Group (Ireland) Ltd

19 Baggot Street Lower,
Dublin 2, D02 X658, Ireland

E-mail: Ireland@arazygroup.com