Triflex Basket

Instructions for Use

EN

Triflex Basket

Instructions for use

ES

Cesta Triflex

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IT

Cestino Triflex

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Triflex Basket

DEVICE DESCRIPTION

Triflex basket is a flexible device with a long, thin flexible shaft and an expandable basket of wire made of Nitinol at the distal end. It is designed to, grasp, release, manipulate and extract urinary stones from the ureter or the kidney in an endoscopy procedure. It is introduced through the working channel of the endoscope. The adjustable handle allows for controlled and assured basket expansion.

INTENDED PURPOSE & DURATION

Triflex Basket is intended to be used in an endoscopic procedure for removal of urinary stones or foreign objects from the Ureter or the Kidney for a period of less than 60 minutes.

INDICATIONS FOR USE & CONTRAINDICATIONS

Indicated to be used to retrieve urinary stones or foreign objects from ureter or kidney. There are no known absolute contraindications but following is listed but not limited.

- Do not use the basket to grip the stone when its size is too large to remove endoscopically.
- To avoid stone impaction fluoroscopy or x-ray should be used to determine the size of the stone.

POTENTIAL COMPLICATIONS

Complications may occur at any time during or following the procedure. Possible complications include: Oedema, Urosepsis, Perforation of the ureter & Iatrogenic related infections.

WARNINGS

- Manipulation of the device needs ultrasound or fluoroscopic guidance.
- If stone size is too large, break the stone into fragments and then retrieve, otherwise it may cause damage to the basket.
- To avoid stone impaction, fluoroscopy or X-Ray should be used to assess the size of the stone.

Triflex Basket

- The device is conductive. Avoid contact with electrified instrument.
- Do not use if product is damaged or expired.
- Do not use if package integrity is compromised.
- Do not reuse the device. Re use/ Resterlization of the device can compromise structural integrity of the device thereby causing serious harm to the patient. Reuse can contaminate the device causing serious harm to the patient by way of infection.
- Do not open the basket or operate the device handle when the tip of the opening side of the basket is still inside the endoscope. It may lead to failure of the device.

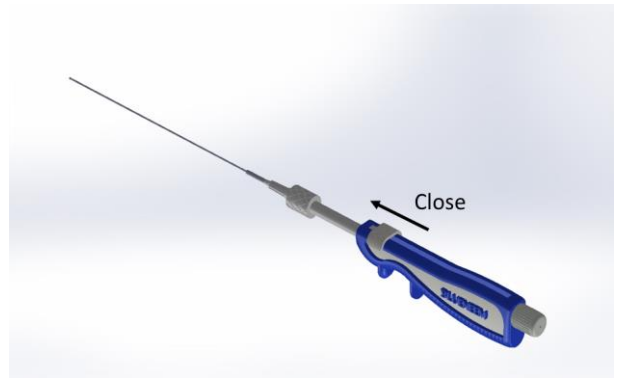
PRECAUTIONS

- Device use is intended to be carried out by trained physicians and technicians who were trained in endoscopic urological procedures. Standard endoscopic procedures should be employed.
- Do not use excessive force to manipulate the device, it may cause damage to the device.
- Not intended for permanent use. Only intended for transient use.

INSTRUCTIONS FOR USE

- Remove the Triflex basket from outer package and move the handle position to “Close” position to close the basket as shown in below figures.
- Insert the device into the scope and under direct visualization, locate the stone and advance the basket to the location just proximal to the stone.
- Slide the handle lever to open position as shown in below figures for different types of handles to deploy the basket near the stone.
- Maneuver the device until the stone is within the basket
- Once correctly positioned, draw the basket down around the stone and operate the handle as shown in below figures.
- Grasp the stone and release it in the upper pelvic junction or retract it through the scope by maintaining it in the position of the basket.

Triflex Basket

SUPPLY AND STORAGE

The device is packed with dispenser and then packed in peel open Tyvek pouch made of polyethylene and Tyvek. Supplied sterilized by Ethylene Oxide gas in peel open package.

Device shall be stored at temperature range from 20⁰C to 40⁰C and away from direct sunlight and other adverse environmental conditions.

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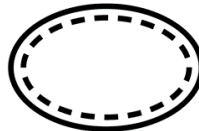
Non-pyrogenic

Apirógeno
Apirogeno

Single sterile barrier system with protective package inside

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Blueneem, the Blueneem Logo are Registered and Trademarks of Blue Neem Medical Devices Private Limited Company.

© 2024 All rights reserved

BLUE NEEM MEDICAL DEVICES PRIVATE LIMITED

Plot Nos 270 & 271, Road No 5,
Harohalli Industrial Area, II Phase,
Kanakapura Taluk, Ramanagara -
562112, India.

Phone: +91 9739972855 / 54

E-Mail: contact@blueneem.com

Website: www.blueneem.com

**Authorized Representative in the
European Community:**

Arazy Group (Ireland) Ltd

19 Baggot Street Lower,
Dublin 2, D02 X658, Ireland

E-mail: Ireland@arazygroup.com