# **Praveza<sup>®</sup> Ureteral Access Sheath**

## **Instructions for Use**

**EN**      **Praveza<sup>®</sup> Ureteral Access Sheath**  
Instructions for use

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# **Praveza® Ureteral Access Sheath**

## **DEVICE DESCRIPTION**

Praveza® Ureteral Access Sheath is a flexible tube-like instrument. It is designed to function as a barrier for the endoscope and other instruments from patient contact. It contains ports/ lumens for the insertion of instruments.

## **INTENDED PURPOSE & DURATION**

Praveza® Ureteral Access Sheath is used to establish a conduit during urological procedures facilitating the passage for Ureteroscope/endoscope and other surgical instruments into the urinary tract. The working channel protects the ureter during repeated instrument exchanges. The sheath also protects delicate instruments and smaller flexible scopes from damage for a period of up to 06 hours.

## **INDICATIONS FOR USE & CONTRAINDICATIONS**

Indicated to be used in urinary obstruction due to urinary stones. Patients who are contraindicated for retrograde urological procedures; presence of tight strictures, which would limit use of the device; presence of large obstructing distal ureteral stone.

## **POTENTIAL COMPLICATIONS**

Potential complications include but not limited to Edema, irritation, inflammation & ureteral avulsion.

## **WARNINGS AND PRECAUTIONS**

- Manipulation of device requires Ultrasound / Fluoroscopy / Other image guiding techniques.
- Do not use if product is damaged or expired and reuse of this device leads to potential complications.
- This device is intended for use by physicians trained and experienced in diagnostic and endoscopic techniques, standard techniques for placement of endoscopic devices should be employed.
- On completion of the procedure, dispose off the device and its components, if any, as per institutional and local regulatory guidelines for bio-hazardous medical waste disposal.
- Not intended for permanent use.
- Never use undue force for placement of this device.
- Use only compatible endoscopic devices.
- Do not re-sterilize the device. It might show loss in structural integrity.
- Refer to the device label, for technical specifications.
- Obturator shall be locked while inserting the sheath through the ureter.
- Do not force the device when resistance occurs while usage.

# **Praveza® Ureteral Access Sheath**

## **INSTRUCTIONS FOR USE**

1. Prior to the placement, activate the hydrophilic coating on the device, by removing the dilator from the access sheath and immersing all components in sterile water or isotonic solution. This will make the sheath lubricious.
2. Place an 0.035” inch guidewire of 150 cm length into the ureter to establish a working tract using standard procedure.
3. Ensure, the dilator is securely locked on to the Sheath adapter, allowing the dilator or sheath assembly to be placed as a single unit.
4. Advance the distal tip of the dilator and sheath assembly over the guidewire into the ureter.
5. Confirm proper placement using fluoroscopic guidance.
6. While maintaining sheath position, unlock the dilator and remove the dilator. Do not advance the sheath without the dilator in place.
7. The desired endoscope or instruments can now be used through the Praveza® Ureteral Access Sheath as needed. On completion of the access procedure, gently withdraw the device.

## **SUPPLY AND STORAGE**

The device is packed with primary pack made of polyethylene and the secondary package is made of polyethylene and Tyvek pouch. Supplied sterilized by Ethylene Oxide gas in peel open package.

Device shall be stored at temperature range from 20<sup>0</sup>C to 40<sup>0</sup>C and away from direct sunlight and other adverse environmental conditions.

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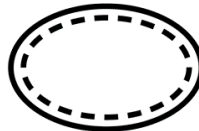
**Non-pyrogenic**

Apirógeno  
Apirogeno

**Single sterile barrier system with protective package inside**

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Blueneem, the Blueneem Logo are Registered and Trademarks of Blue Neem Medical Devices Private Limited Company.

© 2024 All rights reserved

## **BLUE NEEM MEDICAL DEVICES PRIVATE LIMITED**

Plot Nos 270 & 271, Road No 5,  
Harohalli Industrial Area, II Phase,  
Kanakapura Taluk, Ramanagara -  
562112, India.

Phone: +91 9739972855 / 54

E-Mail: [contact@blueneem.com](mailto:contact@blueneem.com)

Website: [www.blueneem.com](http://www.blueneem.com)

---

**Authorized Representative in the  
European Community:**

**Arazy Group (Ireland) Ltd**

19 Baggot Street Lower,  
Dublin 2, D02 X658, Ireland

E-mail: [Ireland@arazygroup.com](mailto:Ireland@arazygroup.com)