Amplatz Renal Dilators

Device Description

Amplatz Renal Dilators are tube like device made of medical grade plastic material, with a long slender hollow body and a tapered tip at the distal end. They are designed for stretching or enlarging the fascial tissue covering a cavity, tract or opening during a procedure. The Dilators are radiopaque for fluoroscopic guidance. Refer to the device label for product specifications.

Size (Fr)	Length (cm)
12, 14, 16, 18, 20, 22, 24, 26, 28 & 30	Amplatz Sheath – 17 Renal Dilator - 30

Intended Purpose

Used for dilation of a percutaneous tract and setup of a working channel into the kidney for removal of urinary stones.

Indications For Use & Contraindications

None known.

Potential Complications

Potential complications Include but not limited to Urinary tract infection & Hemorrhage.

Warnings And Precautions

- Use with caution when advancing the sheath and dilators to avoid tissues and orifice damages. Use only compatible advancing sheath.
- Do not use if product is damaged or expired, reuse and resterilization of this device leads to loss in structural integrity and performance. Device is intended for single use only.
- Imaging guidance is required for use of this device.
- This device is intended for use by trained physicians and this device should not be used without comprehensive knowledge of the indications, techniques and risks of the procedure.
- On completion of the procedure, dispose off the device as per local regulatory guidelines for bio-hazardous medical waste disposal.

Instructions of Use

- Perform initial access with puncture needle, place a guidewire and perform dilation as appropriate using standard procedure.
- Now introduce the dilator of required size. Advance to the desired location using imaging guidance.
- Introduce the Amplatz sheath over the dilator, use the amplatz sheath of appropriate size or use the same amplatz sheath if supplied along with the dilator.
- Remove the dilator, leaving the amplatz sheath in position. This can now be used as a working channel to perform the procedure.

NOTE: This Device is detectable by X-Ray or other means of fluoroscopy.

Target Population Group

Pediatric and Adult Male and Female population groups.

Supply and Storage

The device is packed in primary pack made of polyethylene and the secondary package made of polyethylene and Tyvek. Supplied sterilized by Ethylene Oxide gas in peel open package. Device shall be stored at temperature range from 20°C to 40°C and away from direct sunlight.

Report Any Serious Incident

If problems occur using this device or any serious incidents or adverse events, please contact our customer relations department or call to the contact provided in our manufacturing address and competent authority of the Member State in which the user and/or patient is established.

Symbol Glossary And Definitions

Standard Reference

ISO 15223-1 Medical devices - Symbols to be used with information to be supplied by the manufacturer

Symbol	Symbol Title
	Do not re-sterilize
	Do not use if package is damaged and consult instructions for use
	Consult instructions for use
	Temperature limit
	Keep away from sunlight
	Do not re-use
	Not made with natural rubber latex
	Sterilized using ethylene oxide
	Non-pyrogenic
	Keep dry
	Medical device
	Single sterile barrier system with protective packaging inside
	Does not contain DEHP
	Unique device identifier
	Date of Manufacture
	Use-by date
	Batch code

	Catalogue number
	Authorized Representative in the European Community/ European Union
	Manufacturer
	Caution
	Prescription

- This device is to be sold by/on the directions of a physician. Sterile (ETO) if package is unopened or undamaged.
- Read all the instructions before use.
- On completion of the procedure, dispose off device and its components if any as per institutional and local regulatory guidelines for bio-hazardous medical waste dispose.

Authorized Representative in the European Community
 Arazy Group (Ireland) Ltd
 19 Baggot Street Lower, Dublin 2,
 D02 X658, Ireland
 E-mail: Ireland@arazygroup.com

Manufacturer
Blue Neem Medical Devices Pvt Ltd
 Plot Nos 270 & 271, Road No 5, Harohalli Industrial Area
 II Phase, Kanakapura Taluk, Ramanagara
 Karnataka - 562112, India.
 Phone: +91 80 29761336
 E-Mail: contact@blueneem.com
 Website: www.blueneem.com

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