

Cath tap

Device description

Cath Tap is a tap like device, which can be connected to foley catheters, which can operate to control the flow of urine from the catheter to outside.

Intended Purpose & Duration

Intended to be used for intermittent drainage of foley catheter.

Contraindications

No known contraindications.

Warnings And Precautions

- Use on or before the expiration date.
- Do not use if package is opened or damaged. Do not use the product if there is suspect that the product is not sterile.
- Upon completion of intended purpose, dispose of the device and its components if any as per local regulatory guidelines for bio-hazardous medical waste or as informed by physician during training.

Instructions For Use

1. After inserting of foley catheter, connect the Cath Tap to the drainage port of the catheter.
2. And cath tap can be connected to urine bag also for collection of urine.
3. Turn the knob on the device to open position, when ever drainage from the catheter is required.
4. Turn the knob on the device to close position to stop the drainage from the catheter.

Intended population group

Adult male population who have temporary urinary incontinence.

Supply and Storage

The device is packed in medical grade pouch made of polyethylene and Tyvek. Supplied sterilized by Ethylene Oxide gas in peel open package. Device shall be stored at temperature range from 20°C to 40°C and away from direct sunlight.

Report Any Serious Incident

If problems occur using this device or any serious incidents or adverse events, please contact our customer relations department or call to the contact provided in our manufacturing address and competent authority of the Member State in which the user and/or patient is established.

Symbol Glossory And Definations

Standard Reference:

ISO 15223-1 Medical devices — Symbols to be used with information to be supplied by the manufacturer

Symbol	Symbol Title
	Do not re sterilize
	Do not use if package is damaged and consult instructions for use
	Consult instructions for use
	Temperature limit
	Keep away from sunlight
	Do not re-use
	Not made with natural rubber latex
	Sterilized using ethylene oxide
	Non-pyrogenic
	Keep dry
	Medical device
	Single sterile barrier system without protective packaging inside
	Does not contain DEHP
	Unique device identifier
	Date of Manufacture
	Use-by date
	Batch code

	Catalogue number
	Authorized Representative in the European Community/ European Union
	Manufacturer
	Caution
	Prescription



- This device is to be sold by/on the directions of a physician. Sterile (ETO) if package is unopened or undamaged.
- Read all the instructions before use.
- On completion of the procedure, dispose off device and its components if any as per institutional and local regulatory guidelines for bio-hazardous medical waste dispose.



Authorized Representative in the European Community
Arazy Group (Ireland) Ltd
19 Baggot Street Lower,
Dublin 2, D02 X658, Ireland
E-mail: Ireland@arazygroup.com



Manufacturer
Blue Neem Medical Devices Pvt Ltd
Plot Nos 270 & 271, Road No 5, Harohalli Industrial Area
II Phase, Kanakapura Taluk, Ramanagara
Karnataka - 562112, India.
Phone: +91 80 29761336
E-Mail: contact@blueneem.com
Website: www.blueneem.com